IN3air O₂

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Important Safety Instructions

The following instructions are provided to ensure your safety and to protect the unit from accidental damage.

- **Read these instructions** – All the safety and operating instructions should be read before this product is operated.
- **Keep these instructions** – The safety and operating instructions should be retained for future reference.
- **Heed all warnings** – All warnings on the appliance and in the operating instructions should be adhered to.
- **Follow all instructions** – All instructions for use and operation should be followed.
- **Warranty** – included inside.

Warning – This symbol indicates important safety instructions that must be followed to avoid injury.

Caution – This symbol is to bring important information to your attention to avoid the risk of injury or damage to the unit.




Prohibited – This symbol indicates a forbidden action. Adhere to these to reduce the risk of injury or damage to the unit.

Electrical Safety




- The unit should only be connected to a main power supply with voltage that matches that specified.
- To prevent overload, do not share the same power supply socket with too many other electronic components.
- Do not place the power cord where it could be stepped on or tripped over.
- Do not place heavy items on the power cord, they could damage the wire.
- Do not allow water or moisture to enter the unit or power adapter.
- Do not use in wet, moist areas such as bathrooms, steamy kitchens or near swimming pools.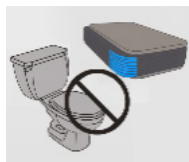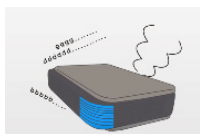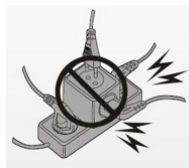

- Pull the plug out immediately and seek professional help if the plug or power cord is damaged, liquid is spilled onto the unit, if it is accidentally exposed to moisture or the unit is not working normally.

Operation Safety

- Keep the product away from flammable gases and liquids, candles and open flames.
- Do not install near any heat sources such as radiators, stoves, or other apparatus that produce heat.
- Do not attempt to repair or replace any part of this equipment unless it is specifically recommended in the user guide. All other services should be referred to a qualified technician.
- Make sure the power cord is plugged firmly and completely into the outlet.
- Hold the main plug, not the wires, when removing from a socket.
- During a thunderstorm switch off the unit and unplug from the socket.
- Do not handle the unit with wet hands.
- Only clean with a soft, dry cloth. Do not use solvents or petroleum based fluids.

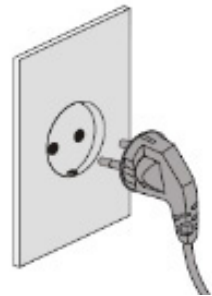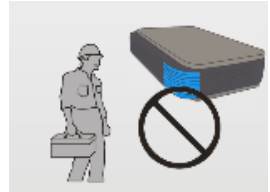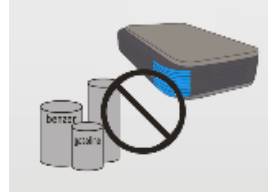- Do not place heavy objects on the unit.
- Do not shake, knock or move the product while in operation, you may damage the internal circuits.

Operation Precautions

IMPORTANT: INBair O2 IS NOT A MEDICAL DEVICE, IT IS FOR RECREATIONAL USE ONLY

If in doubt, consult a medical professional before use.

- **Avoid Excessive use**

Excessive oxygen therapy can be harmful to the human body. The unit will automatically shut down after a usage time of 30 minutes to prevent accidental over-use.

- **Direct the flow of oxygen correctly**

Avoid directing the flow of oxygen towards your eyes. Prolonged airflow directly to the eyes from the headset outlet may cause the eyes to dry out.

- **Handle with care**

The aluminium cover acts as a heat sink and may become very hot after prolonged use. Take care when handling and supervise children at all times.

- **When not in use turn off power and replace outlet cap**

Prolonged exposure of the internal parts to ambient air may compromise the functioning of the unit

- This equipment is not to be used by children or persons with reduced physical, sensory or mental capabilities. Not a toy. Children should be supervised at all times.

What's Included and Optional Parts


Body and Accessory

1. Main Body
2. Headset
3. Spare Filter
4. User Guide
5. Universal Power Adaptor

Optional Parts

6. Shoulder Bag

Key Parts and Components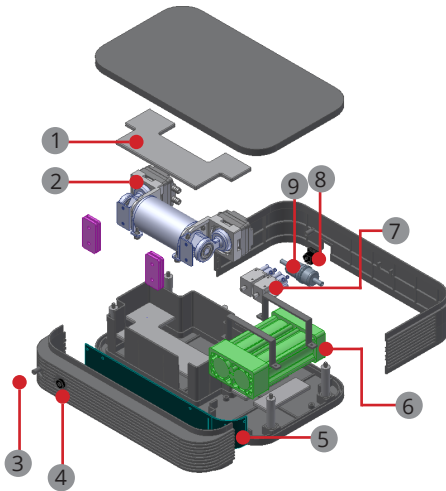

1. Silicone Gel
2. Vacuum Pump
3. Headset Connection (Oxygen Outlet)
4. Power Switch
5. PCB / LED
6. Zeolite Filter
7. Solenoid Valve
8. DC Power Out
9. Internal Air Filter

Operation Instructions

1. Place the headset around the neck and gently direct the flexible mouthpiece towards the mouth and nose.
2. Connect one end of the silicone hose to the oxygen inlet on the headset.
3. Remove the silicone cap from the oxygen outlet on the main body and store in a safe place.
4. Connect the other end of the silicone hose to the oxygen outlet on the main body.
5. Connect the power supply to the back of the product and plug into power socket.
6. Begin the flow of oxygen by gently pressing the power button.
7. After 30 minutes operation time the unit will automatically switch off. Unplug the power and replace the silicone outlet cap to prevent damage.

Caution:

- Always turn off the product by pressing the power button. Unplugging while the unit is drawing power may damage the plug socket and the unit.

Maintenance Instructions


Important

- Disconnect the power before carrying out maintenance

How to Replace the Zeolite Filter

After 600 hours of operation, the LED light will flash, indicating it is time to replace the zeolite filter. Contact your local customer service centre to purchase a replacement part then follow the instructions below:

1. Open the unit as described above and locate the zeolite filter.
2. Loosen the screws on the plastic cover holding the zeolite filter in place and remove.
3. Remove the hose from the zeolite filter and take the component out. Replace with a new zeolite filter.
4. Re-connect the hose into the new zeolite filter as shown below.

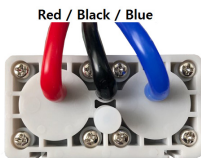5. Replace the plastic cover and tighten the screws to fix the zeolite filter in place. Replace the aluminium cover and tighten the screws to complete reassembly.
6. To reset the automatic timer: Ensure the unit is switched off while plugged into a power socket. Press and hold the power button for 5 seconds. Then press the power button five times.

How to Replace the Vacuum Pump

Important

- Always turn off the product by pressing the power button. Unplugging while the unit is drawing power may damage the plug socket and the unit.

After approximately 1,000 hours (or after the second zeolite filter change) the pump performance may decline, with a noticeable reduction in airflow. To maintain optimum performance of the unit, contact your local customer service centre to purchase a replacement part, then follow the instructions below:

1. Open the unit as described above and locate the vacuum pump.
2. Remove the silicone gel from the top of the pump.
3. Disconnect the four silicone hoses.
4. Remove the cable connecting the pump and the PCB.
5. Lift the pump out and remove the silicone pad from underneath the pump.
6. Remove the four silicone feet and steel brackets from the base of the pump.
7. Attach the four silicone feet and steel brackets to the base of the replacement pump.
8. Replace the silicone pad on to the base of the pump and place it into the main body.
9. Re-connect the cable to connect it to the PCB.
10. Re-connect the four silicone hoses.
11. Replace the silicone gel onto the top of the pump.
12. Replace the aluminium cover and tighten the screws to complete reassembly.

How to Replace the Air Filters

Important

- Disconnect the power before carrying out maintenance

To work effectively the internal and external air filters should be replaced when heavily discoloured, or every 6 months if no discolouration occurs.

Replace the External (Headset) Filter

Remove the hose from both sides of the old filter and reattach them to both sides of the replacement filter.

Replace the Internal Filter

1. Open the unit as described above and locate the internal filter.
2. Remove the hose connected to the filter and replace it with a new filter.
3. Replace the aluminium cover and tighten the screws to complete reassembly.

Troubleshooting

Before you call for service, review this list. It may save you time and expense. This list includes common occurrences that are not the result of defect in workmanship or materials.

Problem – Unit will not turn on.

Possible Cause: Is power cord correctly connected to outlet?

Solution: Ensure power cord is properly inserted into the unit and socket. Check if the socket is providing power

Possible Cause: Check if the voltage of DC adaptor is too low?

Solution: This product should be used with a DC 12V 2.5A adaptor.

Problem –The unit switches off during use

Possible Cause: Was the operation time 30 minutes?

Solution: The unit will automatically switch off after 30 minutes of operation to avoid accidental overuse and over-heating.

Possible Cause: The unit has over-heated due to prolonged use.

Solution: A temperature sensor will detect if the internal temperature of the unit exceeds 55 °C and prompt an automatic shutdown to prevent damage. Unplug the unit and allow to cool before further operation

Problem – Oxygen concentration drops significantly.

Possible cause: Humidity in the ambient air has damaged the zeolite filter.

Solution: Replace the zeolite filter. Always ensure the oxygen outflow on the main body is covered by the cap when not in use.

Problem – The LED lights are not flashing in sequence as usual.

Possible cause: One LED flashing continuously indicates total operation time has reached 600 hours.

Solution: Replace zeolite filter and reset timer as described above.

Problem – Weak wind volume or unusual noise from the unit.

Possible cause: If zeolite filter or the vacuum pump has been replaced, tubes may have not been replaced correctly or come loose.

Solution: Unplug the unit and remove the aluminium cover. Check all tube and wire connections are correct and properly attached.

Possible cause: The operation time has reached 1000 hours and the vacuum pump is due for a replacement.

Solution: Replace the vacuum pump as described in this manual.


Customer services

In this manual, you will find a card containing details of your limited warranty and contact details for your local customer services center, to whom all enquiries and warranty claims should be directed. All warranties are rendered void and the cost of repair will be payable by the user in the case of:

- Damages caused by putting foreign objects (such as water, soft drink, coffee, fragrance oils or toys, etc.) in or on the unit.
- Damages caused by transportation or handling.
- Damages caused by use of products such as benzene or thinner.
- Damages caused by incorrect power supply.
- Breakdown or loss of parts caused by unauthorized disassembling or modification.
- Repair or replacement undertaken by unauthorized persons.
- Damages caused by non-adherence to the instructions provided in this user guide, and as given by the local customer service center.
- Damages caused by careless or improper use.
- Damages caused by operating the product in a corrosive or moist environment.

Specifications

Radic8 reserves the right to make changes to the exterior, design or specifications of the product without prior notice.

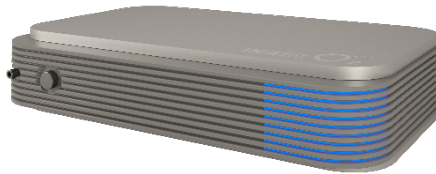Description	Oxygen Purifier
Brand	INBair O2
Model No	O2-100
Elec. Power	DC 12V / 2.5 A
Elec. Consumption	30 watts
Net Weight	2.45 Kg
Gross Weight	3.5 Kg
Net Dimension	281 x 181 x 60.2 mm
Box Dimension	420 x 247 x 130 mm

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Sales & Design: Radic8 Ltd Man & Scythe, 6-8 Churchgate Bolton, BL11HL
Manufacturer: INBair Co., Ltd, 4F, Samyeong Building 21, Samjak-ro